



Strong teeth – with prosthesis!
ANKYLOS® SynCone®



Implants
for life

DENSPLY
FRIADENT

Initial situation:



The bone in the toothless jaw degenerates and the prosthesis “jiggle.”

ANKYLOS® SynCone® gives you that secure feeling

Tooth implants which are firmly integrated in the jaw enable secure holding of prosthesis. They will provide a stable fit for your existing prosthesis.

These days implant treatment is a simple and gentle approach. ANKYLOS® SynCone® enables dentists to offer a proven and cost-efficient treatment option. In many cases you will be able to leave the dental practice in only two hours after implantological procedure – with your own securely fixed prosthesis – and will soon be able to enjoy life as before.

Ten frequent questions – ten convincing answers:

1. What is ANKYLOS® SynCone®?

With ANKYLOS® SynCone® your existing denture is fixed to implants in the toothless lower or upper jaw using tapered retaining elements. SynCone® provides a highly secure fit of your dentures and gives considerable comfort while chewing or speaking. The prosthesis is made as small as possible and feels like real teeth. And yet you can continue to remove your dentures for easy cleaning.

Result:



Lower jaw prosthesis on four implants.



The prosthesis are held in place securely.

2. How will my prosthesis be attached to my jaw?

Four implants will be placed in the frontal region of the mandible and six implants in the maxilla while using a local anesthetic. Your prosthesis is fixed using abutments which are screw-retained to the implants.

3. How long does implantological procedure take?

Your prosthesis is fitted securely immediately after implantation, while the anesthetic is still in effect. After approximately two hours you can leave the dental practice with your dentures firmly in place.

4. Are my jaws suitable for this type of restoration?

As a rule, an radiographic image can determine suitability. Should your bone situation not allow for immediate restoration, it is possible to let the implants heal for three to six months and then use a reinforced prosthesis. During the healing time you continue to wear your existing dentures as usual.

5. Will my jaw bones tolerate the implants?

ANKYLOS® implants are made of medical grade pure titanium. Incompatibilities or allergies have not been observed. There are only very few underlying diseases which would contradict the use of implants.



6. What must I pay attention to immediately after implantation?

Initially the fitted prosthesis must be worn permanently for two weeks. During this time a soft diet and avoiding physical loading are recommended.

7. What happens if an implant does not heal?

The success rate for SynCone® is 97.5 percent.* However, should an implant not heal, your dentist can place a new one.

8. For how many years do these implants last?

With good oral hygiene and regular dental check-ups, prostheses on ANKYLOS® SynCone® are permanently trouble-free.


9. What does this type of implant restoration cost?

Compared with other implant-supported restorations of edentulous jaws, ANKYLOS® SynCone® is a cost-efficient solution.

10. Where do I get ANKYLOS® SynCone®?

Ask your dentist or ask our patient hotline for information: +49 1805 167788.

* May D., Romanos G.E. in „Die Quintessenz“, March 2001

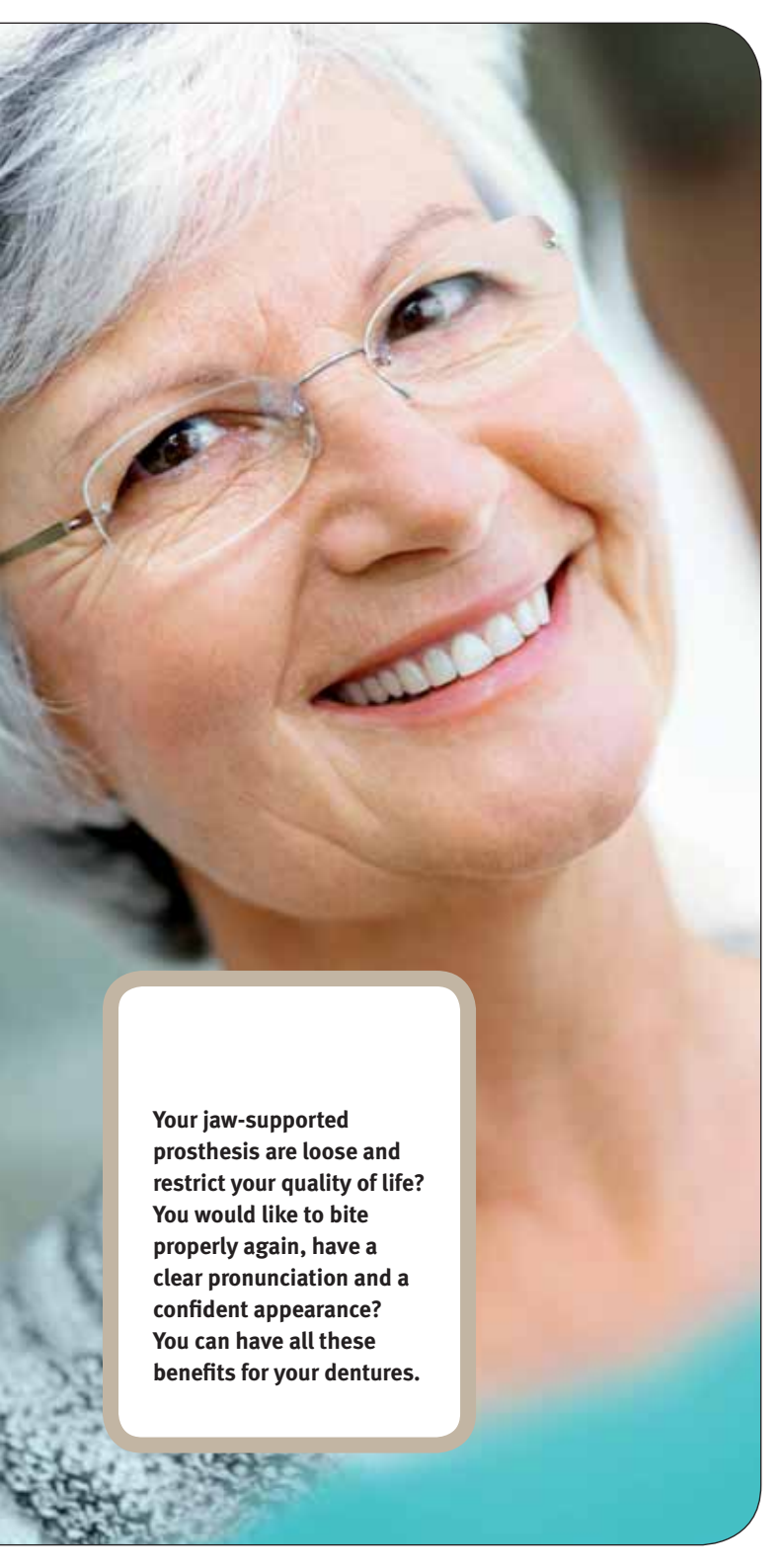

**Your jaw-supported
prosthesis are loose and
restrict your quality of life?
You would like to bite
properly again, have a
clear pronunciation and a
confident appearance?
You can have all these
benefits for your dentures.**

ANKYLOS® SynCone® – for all cases

Even if immediate fitting of the dentures should not prove possible, ANKYLOS® SynCone® will be able to provide comfortable and permanent fixing after a certain healing time of the implants.

Do you still have questions to your dentist?

Your dentist will be glad to give you comprehensive answers to your specific questions without any obligation and will give you information on the treatment procedure and the costs involved.



Practice stamp

Order no. 5-252014/002

DENTSPLY Friadent | DENTSPLY Implants Manufacturing GmbH
P. O. Box 71 01 11 · 68221 Mannheim/Germany
Phone +49 621 4302-1265 · Fax +49 621 471306
e-Mail: friadent@dentsply.com

www.implantsforlife.com

DENTSPLY
FRIADENT

0712/5.0/GVD